# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy

Italia: tel. 199 400 401 - fax 199 400 403

Export: tel. +39 02 953854209/221/225 fax +39 02 95380056

[gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [export@gimaitaly.com](mailto:export@gimaitaly.com)

[www.gimaitaly.com](http://www.gimaitaly.com)

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**Height measure**

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### FEATURES

GIMA tape height measure (cod. 27335) is an instrument suitable to measure a person's height with an approximation of  $\pm 1$  cm.

It can be secured to the wall and measure up to 2 m. It is space-saving and allows a quick reading.

### PRESCRIPTIONS

*Do not pull the slider abruptly, as the graduated tape could break.*

The product shall be used by qualified personnel only.

*Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

### UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## INSTALLATION

In order to assemble the height measurer, the cursor should be positioned on the wall exactly at a height of 2 metres in line with the surface base and, keeping the product compact, fix the small plate using the relative pins. At this point the height measurer is ready for use.

## OPERATION

In order to measure height simply take the cursor with one hand and pull it downwards until it touches the head of the person to be measured, then simply read the measurement in correspondence with the red bar in the window.

## MAINTENANCE

The product has been designed to last a long time. We recommend that the instrument is cleaned with a delicate detergent.

*Periodically clean the tape height measure with a cloth dampened with disinfectant. Pay particular attention to the slider.*

## GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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GIMA S.p.A.