



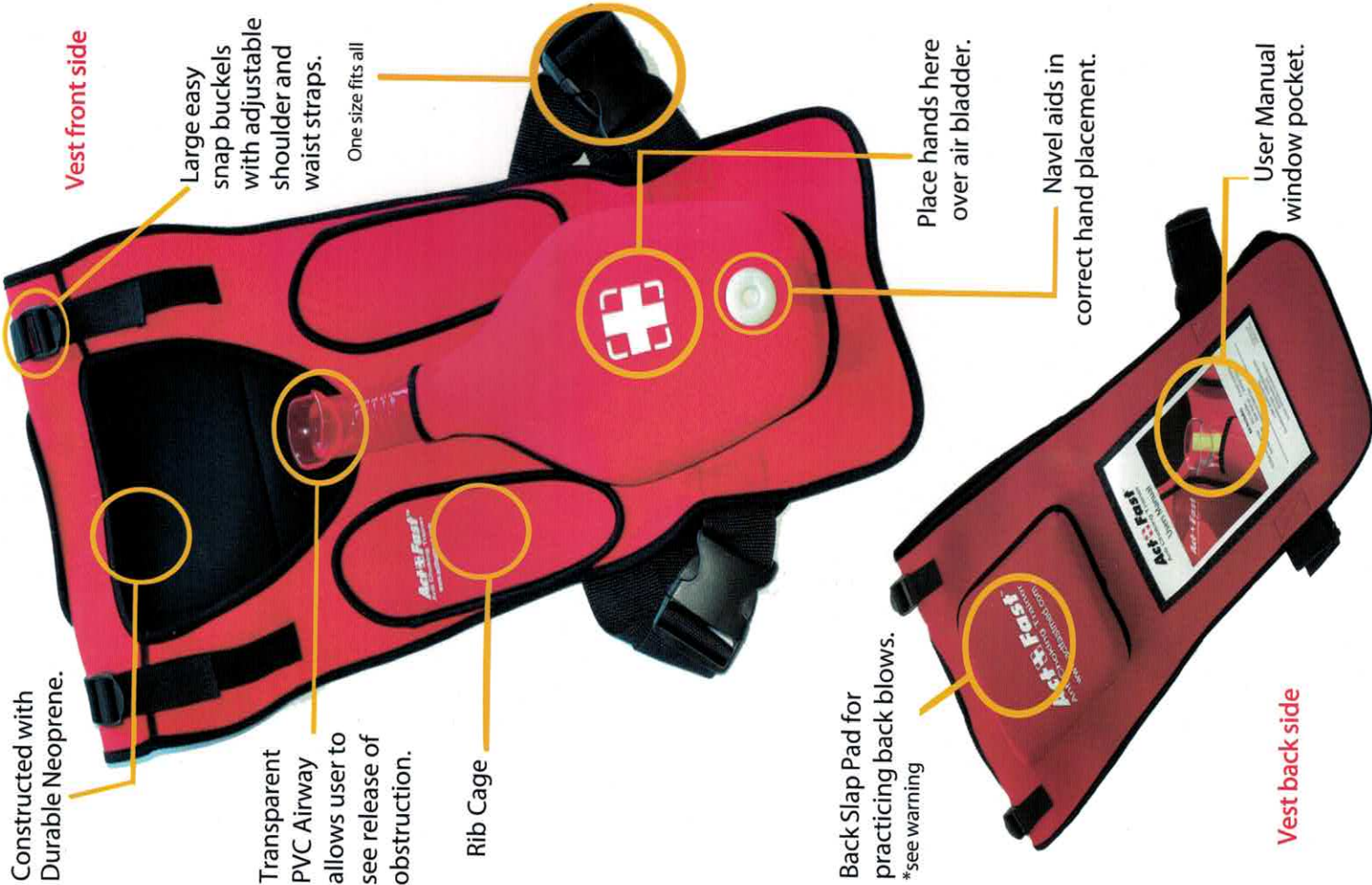

Act+Fast™

Anti Choking Trainer

User Manual

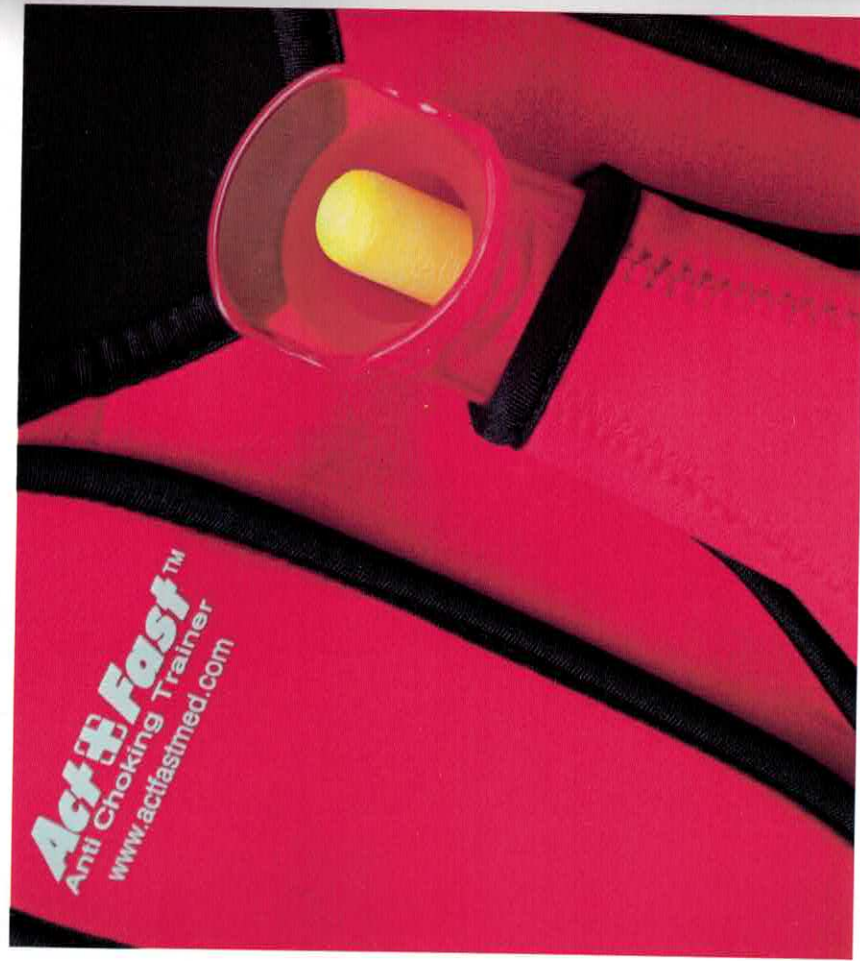

Kit Includes:

- Act+Fast neoprene vest - PVC bladder - Airway assembly
 - Thirty foam plugs - User manual - Storage bag
- All contents are latex-free*

1 year product warranty
warranty@actfastmed.com

MADE IN CHINA


Directions for Use

Intended for use under the direction of certified CPR Instructors.


Pull the vest over your head and cinch the waist belts snug.

Use the Back Pad to practice giving back blows.



WARNING: for simulation only, do not use force.

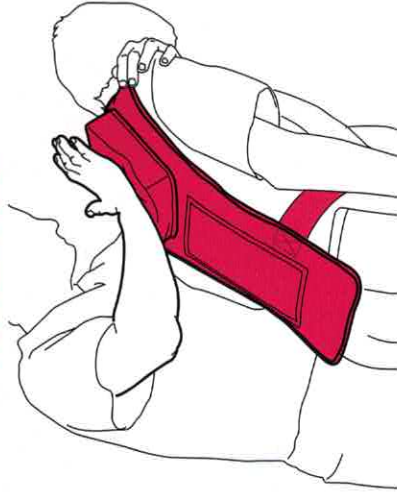

Drop a foam plug into the airway.

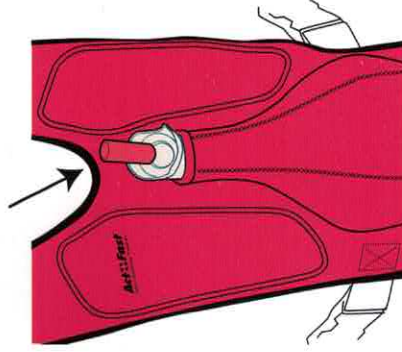

Place your hands in the correct position, between the navel and the rib cage, resting them lightly on the vest.

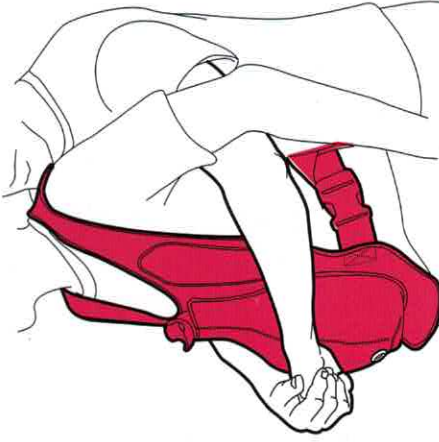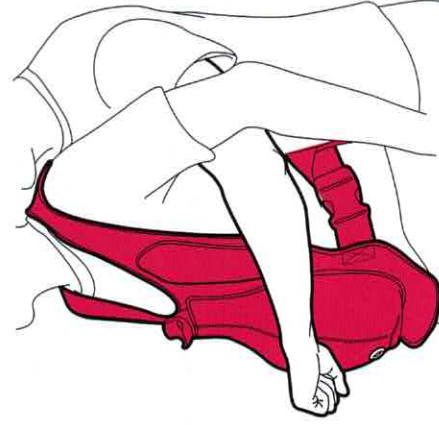

Then deliver a quick upward thrust to dislodge the obstruction. Do not use force, it is not necessary and may cause injury. Correct technique will result in the plug shooting into the air, providing instant feedback and teaching the concept.




Warning: do not aim foam plug at eyes.

The training vest can be used in a variety of positions to create realistic scenarios, including: standing, sitting and laying on the ground. Also, it allows the student to practice self rescue, using a counter top or chair.

Cleaning Instructions

From the underside of the vest, release velcro straps and remove the PVC bladder and airway assembly from neoprene vest.

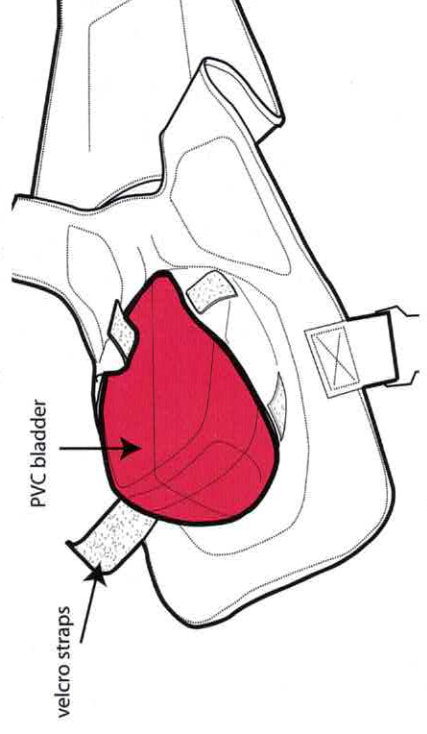

Wash vest by hand using mild detergent, hang to air dry.