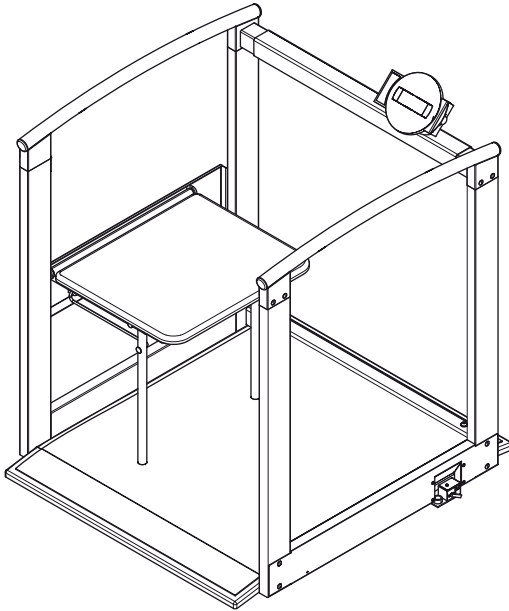
seca 685

D	<i>Bedienungsanleitung und Garantieerklärung</i>	3
GB	<i>Instruction manual and guarantee</i>	25
F	<i>Mode d'emploi et garantie</i>	47
I	<i>Manuale di istruzioni e garanzia</i>	69
E	<i>Manual de instrucciones y garantía</i>	91
DK	<i>Betjeningsvejledning og garantibevis</i>	113
S	<i>Bruksanvisning och garanti</i>	135
N	<i>Bruksanvisning og garantierklæring</i>	157
FIN	<i>Käyttöohje ja takuu</i>	179
NL	<i>Bedieningshandleiding en garantieverklaring</i>	201
P	<i>Instruções de utilização e declaração de garantia</i>	223
GR	<i>Οδηγίες χειρισμού και εγγύηση</i>	245

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Full certification

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes. seca products comply with European directives, standards and national laws. With seca, you are buying the future.

M

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol comply with the following directives and standards:

1. Directive 90/384/EEC governing non-automatic weighing instruments
2. Directive 93/42/EEC governing medical devices
3. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the development, production, sales and service of medical scales and measuring systems and ergometry.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

By purchasing the **seca 685** multifunctional scales you have acquired a highly accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 685** multifunctional scales is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scales are a Class I medical product, and have been calibrated in accordance with the precision class III. The folding seat integrated into the robust handrail makes the **seca 685** suitable not only for weighing patients seated in a wheelchair but also for weighing people who easily lose their balance.

The **seca 685** has two weighing ranges

which can be selected. Weighing range 1 has a reduced maximum capacity but greater display resolution, whilst weighing range 2 allows you to use the maximum load capacity of the scale. Weight is determined within a few seconds.

The **seca 685** is designed to measure weight conventionally but can also determine body mass index. A membrane keyboard allows body size to be entered for obtaining the body mass index that corresponds with weight.

Using the Pre-Tare function, you can store the weight of a wheelchair. This weight is automatically deducted in the weighing operation.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 685** can be moved on castors.

2. Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Do not drop the scale or subject it to violent shocks.
- After transporting the scales, place the scales carefully back in the position for use.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 44).
- Have repairs carried out exclusively by authorised persons.

3. Before you start...

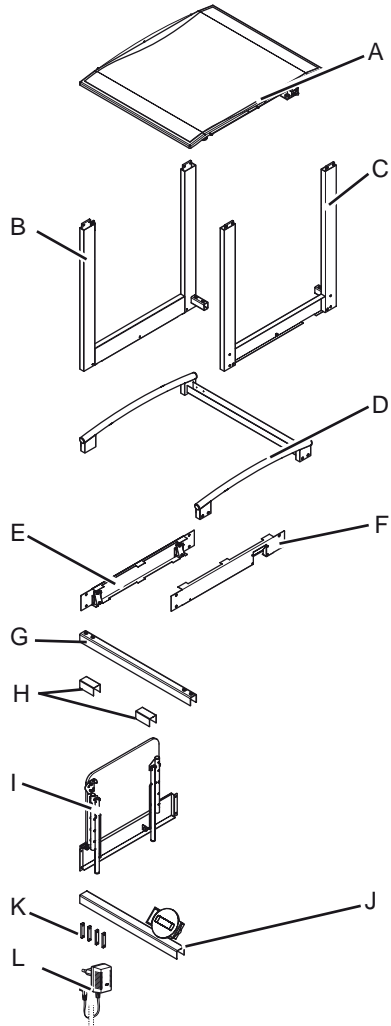
Unpacking

Remove the packing material and place the weighing platform onto a solid, flat surface.

Equipment supplied

The equipment supplied includes:

- 1 weighing platform (A)
- 1 side element left (B)
- 1 side element right (C)
- 1 handrail (D)
- 1 cover left (E)
- 11 cover right (F)
- 1 bottom transverse bar (G)
- 2 covers (H)
- 1 foldable seat (I)
- Display head with support (J)
- 4 threaded anchor plates (K)
- 1 seca power supply unit (L)
- Operating instructions with a declaration of conformity


Bolts and tools

The following small parts and tools are included in the supplied equipment:




- 16 galvanized M6 x 12 countersunk bolts
- 8 black covers for M6 countersunk bolts
- 6 galvanized J 6.4 crown gears
- 4 galvanized M8 x 20 oval headscrews
- 4 galvanized 8.4 washers
- 8 white covers for M6 countersunk bolts
- 4 galvanized B 3.5 x 9.5 pan-head tapping screws
- 4 galvanized 6.4 washers
- 4 galvanized M6 x 20 oval head screws
- 1 SW4 hexagon wrench
- 1 SW5 hexagon wrench
- 1 size 1 Phillips-head screwdriver
- 1 SW 8/10 socket wrench
- 1 SW 10 combination wrench

Assembling the scales


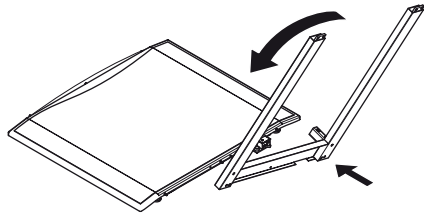
1. Unscrew the three nuts from the bolt on the sides of the weighing platform.
2. Place the right element on the rear bolt of the weighing platform.
3. Swing the side element back behind the level support.
4. Place the side element onto the middle and front bolt of the weighing platform.
5. Place the crown gear and nut onto the middle bolt and tighten it firmly by hand.

6. Tip the scales forward and place it onto the side piece (1).
7. Assemble the rear and front nut together with the crown gear.
8. Tighten the three nuts using the supplied socket wrench and combination wrench. Use the supplied screwdriver as a toggle in the combination wrench.
9. Place the scales back onto the weighing platform.
10. Assemble the side piece accordingly. Tip the scales over after tightening the middle nut.
11. Remove the adapter between the six bolt heads and the frame weighing platform on both sides.
12. Fasten the left (2) and right (3) side piece covers. At each corner, fit two countersunk bolts (4) M6 x 12 and fit the threaded anchor plate (5).
13. Place the scales back onto the weighing platform.
14. Place the handrail (6) onto the two side elements.
15. Use the countersunk bolts M6 x 12 to fasten the handrail into place.
16. Fasten the bottom transverse bar (7) with the M8 x 20 oval head screws and plain washers.

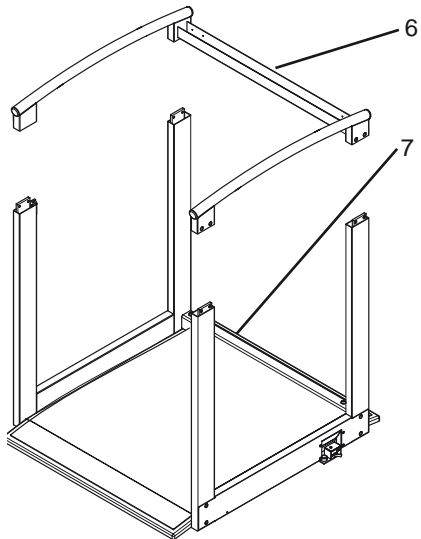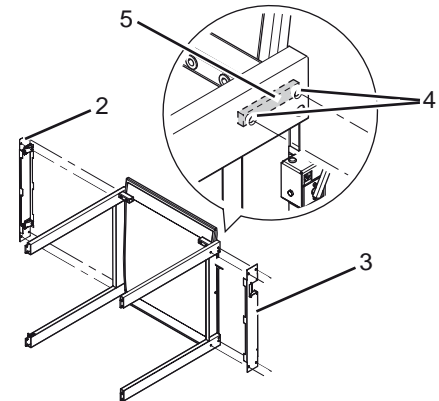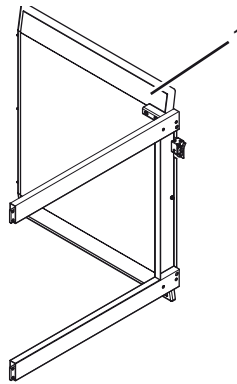
17. Pass the connector cable for the display downward, from the top right through the right rear side post (8).
- Note:** Do not remove the cable tie on the connector cable. When laying the connector cable, make sure it is not pinched or twisted.
18. Place the covers (9) on the left and right end of the transverse bar.
19. Initially fasten the covers on the two outer holes using the B 3,5 x 9,5 pan-head tapping screws.
20. Place the support and display (10) onto the upper transverse bar. Next, fit the bolts on the inner holes of the covers.
21. Connect the connector cable to the electronics box (11).
22. Screw-tighten the folding seat (12) from the rear onto the left side piece using the M6 x 20 oval head screws and plain washers.
23. Remove the cable ties from the legs of the folding seat.
24. Place the appropriate covers on the countersunk bolt heads.

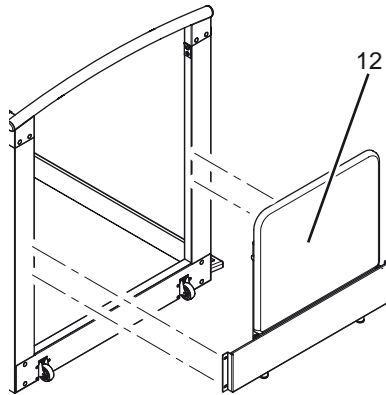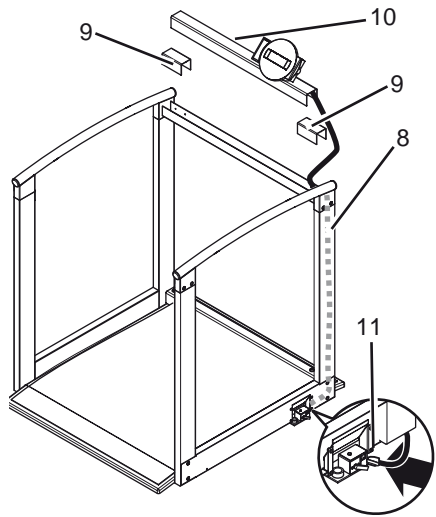Power supply


The scale is powered by a mains unit.

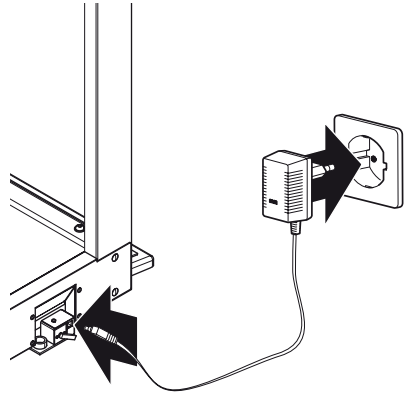Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.

4. Setting up and aligning the scale

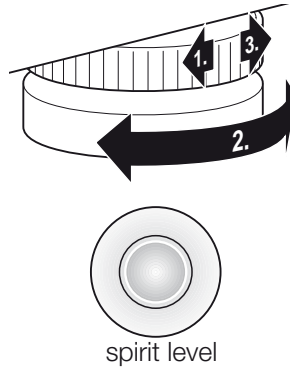- Place the scale on a level surface.
- Undo the locknuts using the four foot screws (1).
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.


Important!

The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.

Moving the scales

To move the scales to a different location,

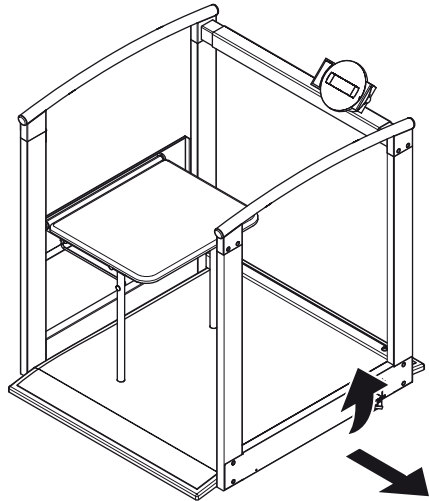
1. Take hold of the left side element and tip the scales carefully over until the scales are on the castors.
2. With the scales in this position, push to the desired location.

5. Operation

Controls and displays

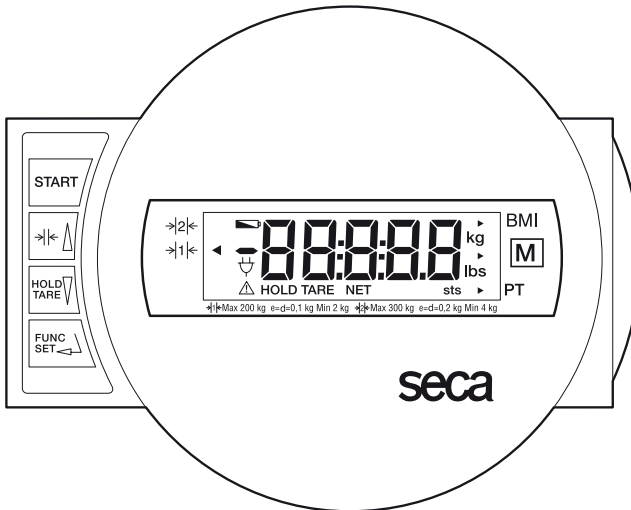The controls and display are in the display part.

Switch scale on and off

press briefly → switch between weighing range 1 and 2
in function menu → increase value, scroll

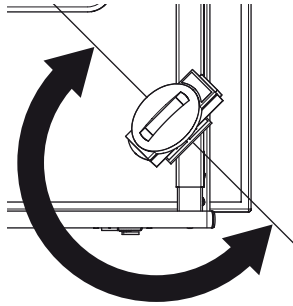
press briefly → activate hold function
hold down → set display to zero, activate/deactivate tare function
in function menu → decrease value, scroll

Menu selection, confirmation, setting up parameters

Turning the display

The display for the scales is mounted facing the chair. The display can be turned 180° so that either the patient or health care personnel can view or operate the display. Turn the display anticlockwise until it is in the desired position.

Folding the seat up or down

To lower the folding seat:

1. Hold the folding seat firmly with one hand.
2. Unfasten the bolt (1) on the left, above the folding seat by turning it clockwise.
3. Lower the seat.

Caution!

Push the bolt beneath the seat so it does not extend out.

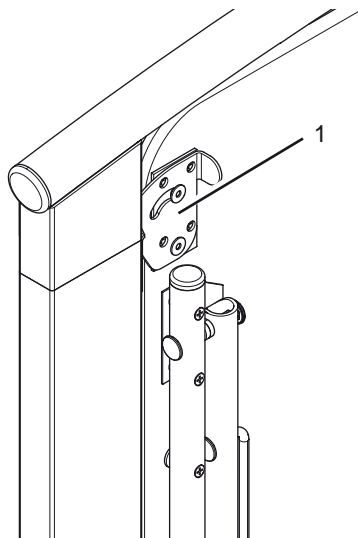
Only load the seat with a weight up to the permissible highest level (150 kg).

To raise the folding seat:

1. Lift the folding seat up.
2. Fasten the folding seat by turning the bolt counterclockwise.

Make sure that the seat is correctly locked into place.

Make sure that the seat is not accidentally unlocked.

Displaying time

After the scale has been switched on, time is displayed for about 2 seconds.

If a printer is connected, the time and date of the weighing operation is printed out in addition to the weight itself.

Depending on the country to which it is supplied, the display can be in either 24-hour clock format (e.g. 19:45) or in 12-hour format (e.g. P 7:45).

If **TIME** appears in the display and 3 alarm signals are heard after the scale is switched on, this means that the scale has been without a power supply for a prolonged period, so the clock needs to be reset.

Setting the clock

- To reach setting mode, press the **FUNC/SET** key for about 2 seconds.

Depending on the version of the scale, the year will appear immediately or additional menu items will appear.

If additional menu items appear, use the arrow keys to select the **TIME** menu item and then press the **FUNC/SET** key to reach the settings menu. The year appears in the display.

- Use the arrow keys to set the correct year.

- Press the **FUNC/SET** key to confirm the year information entered. This simultaneously switches you to the next value you can set. Change this value too using the arrow keys and confirm the entry using the **FUNC/SET** key.

Set all the values for the clock correctly in the same way. The sequence of values is year, month, day, hour, minute.

On clocks with a 12-hour display, **A** appears on the left of the display for morning times and **P** for afternoon and evening times.

- After you have confirmed the minutes using the **FUNC/SET** key, you exit the settings menu automatically and the clock continues to run at the time set.

If you also require seconds to be set accurately, then proceed as follows:

- set a minute which is still to come, but do not confirm it using the **FUNC/SET** key. As soon as the set minute starts, press the **FUNC/SET** key. At that point, the clock will start running at 00 seconds and the set minute.

Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

Both numbers must match.

Correct weighing


- Press the Start key with no load on the scale.
SECA, **88888**, and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- Switch weighing range if necessary (see page 37).
- Stand on the scale and keep still.
- Read the weighing result off the digital display.
- If a load greater than 200 kg or 300 kg is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

Switching the weighing range

The **seca 685** has two weighing ranges which can be selected. Weighing range 1 has a reduced maximum capacity but greater display resolution, whilst weighing range 2 allows you to use the maximum load capacity of the scale. Depending on the application in question, you decide whether you would prefer to use the fine display resolution or the high load range.

- Switch on the scale at the green Start key. The scale initially operates in the weighing range last selected.
- To change the weighing range, press the key $\rightarrow| \leftarrow \blacktriangle$.
- A triangle in the display indicates the weighing range active in each case. This remains active even when the scale is turned off.

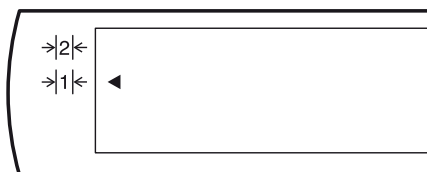Note:

It is possible to switch from Range 1 with fine graduations to Range 2 with coarse graduations with a load on the scale unless the scale is switched to a special function (like pre-Tare, for example). For technical reasons, it is only possible to switch from coarse Range 2 to fine Range 1 with no load on the scale. In this case too, no special function may be activated.

Switching off the scale

- To switch off the scale, press the **START** key again.

Taring weight display (TARE)

The Tare function is very useful if an additional weight, e.g. that of a wheelchair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the Start key with no load on the scale.

- First place the additional weight (e.g. the wheelchair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero, **0.0** flashes in the display until weight is detected to be stable. Only then is the Tare function fully activated. The note "NET" appears.

- Now move the patient and the wheelchair previously weighed onto the scale.

The scale determines the weight of the additional load. You can now perform as many weighing operations as you like. Hold the key down again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.

- The tare value is likewise erased by the scale being switched off.

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value is stored until the scale switches off automatically.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

00 appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

Note:

The HOLD and TARE functions can also be used simultaneously.

Body-Mass-Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health.

- Press the Start key with no load on the scale.

SECA, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key. The last function used appears in the display.

- Use the arrow keys to select the BMI function **FU 1**.

- Activate the function by pressing the **FUNC** key once again.

The patient's recorded height will then appear on the display.

- You can use the arrow keys to adjust the value in increments according to the graduations displayed.

- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated.

- Ask the patient to mount the scale. Once on the scale, the patient should stand still.

- Read off the BMI on the digital display and compare it with the categories on page 40.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Startkey again.

Evaluating Body Mass Index for adults:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**

Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.

- **BMI between 18.5 and 24.9:**

The patient is of a normal weight. He can stay as he is.

- **BMI between 25 and 30 (pre-obese):**

The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function is useful, for example, if the weight of a wheelchair is not to be displayed.

- Press the Start key with no load on the scale.
SECA, **88888** and **0.0** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.
- Press the **FUNC** key. The last function used appears in the display.
- Use the arrow keys to select the pre-Tare function **FU 3**.
- Activate the function by pressing the **FUNC** key once again. In the display you will see the tare value from the memory.
- You can use the arrow keys to adjust the value in increments according to the graduations displayed.
- Once the correct value is set, press the **FUNC** key again. The pre-Tare function is now activated.
- Move onto the scale in the wheelchair associated with that memory. Read the weighing result off the digital display.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.

Tip:

Note weight on your wheelchairs.

Setting damping

Damping is useful for reducing weight determination malfunctions (due to patient movements, for example). The stronger the damping, the more slowly the weight display reacts. You can set damping to suit requirements.

- Press the **FUNC** key and keep it depressed for 2 seconds.

FIL and the current setting appear in the display:

- 0 = slight damping
⇒ fast determination of weight
- 1 = average damping
⇒ normal determination of weight
- 2 = strong damping
⇒ slow determination of weight

- Press an arrow key if you want to change the set value.

- Press the **FUNC** key again to store the displayed value. The scale is then back in normal weighing mode.

The settings are retained even once the scale is switched off.

6. Cleaning

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

7. What do I do if...

... no weight display comes on?

- Is the scale switched on?
- Is the plug contact of the mains unit properly connected to the scale?
- Is the plug contact for connecting the display to the scale correctly connected?

... 0.0 does not appear before weighing?

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

... 5tOP appears in the display?

- Maximum load has been exceeded.

... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until 0.0 is displayed, then weigh again.

... the display tENP appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

... the display E and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.

If this is not the case, disconnect the power supply by briefly removing the batteries. If this measure is equally unsuccessful, inform the Service department.

... the weighing range won't switch?

- Take the load off the scale if necessary.
- Ensure that no special function is active.

8. Servicing/recalibration

Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0109 (Hessen Weights & Measures Office). Recalibration is essential if one or more security marks are damaged or the calibration counter does not match the number on the applicable calibration counter mark (cf. page 36).

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

9. Technical data

Dimensions scale

width:	910 mm / 35.8 ins
height:	1100 ⁺¹⁰ mm / 43.3 ^{+0.4} ins
depth:	965 mm / 38 ins

Dimensions weighing platform

width:	800 mm / 31.5 ins
height:	46 ⁺¹⁰ mm / 1.8 ^{+0.4} ins
depth:	850 mm / 33.5 ins

Weight

approx. 59 kg

Temperature range

+10 °C to +40 °C

Calibration as per Directive

90/384/EEC

approved for medical use, Class III

Medical product as per Directive

93/42/EEC

Class I

Colour

white/black

Maximum load

Weighing range 1	200 kg
Weighing range 2	300 kg

Maximum load for folding seat

150 kg

Minimum load

Weighing range 1	2 kg
Weighing range 2	4 kg

Graduations

Weighing range 1	100 g
Weighing range 2	200 g

Accuracy at first calibration

Weighing range 1:

to 50 kg:	± 50 g
50 kg to 200 kg	:± 100 g

Weighing range 2:

0 to 100 kg:

± 100 g

100 kg to 300 kg:

± 200 g

Power supply

mains unit

10. Spare parts

Access ramp

seca spare part no.: 470-00-00-009

RS232 adapter kit with print port

seca spare part no.: 460-00-00-004

Model dependent mains unit

Description: plug-in mains unit

230 V~/50 Hz/12 V~/130 mA

seca spare part no.: 68-32-10-252

Description: desk-top mains unit, switchable

115 V~/230 V~/50 – 60 Hz/9 V~/350 mA

seca spare part no.: 68-32-10-243

Supplementary calibration mark

seca spare part no.: 14-05-01-886

11. Disposal

Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national

regulations which apply in your case. For further information, contact our service department at:

service@seca.com

12. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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